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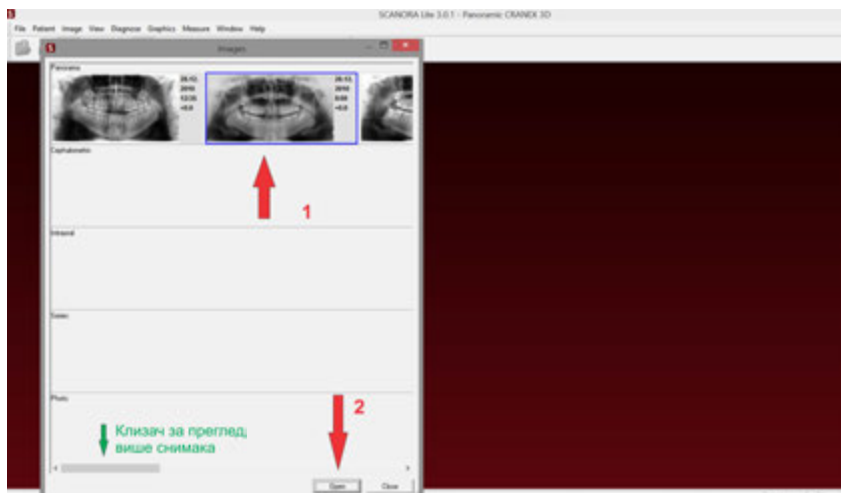
**Scanora Lite** омогућује штампање снимака. Кликом на иконицу штампача (слика 3 на претходној страни) отвара се „Print Editor“. The print editor will print open images as they are seen on screen. If drawings or measurements are visible, those will also be printed. Image information and notes can be added to paper with image. The print editor has tools such as zoom, fit to page, restore image size, page setup and, setup dialog for header and footer. With page setup user can change the paper type, width, height, orientation and measurement units. By using header and footer setup you can define information text for the header and/or footer. Items to be added are e.g. patient's name, patient's ID or date and time.

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**Scanora Lite** has a copy image to clipboard -feature. Images can be copied to the clipboard either as shown on the screen or with their actual size. Image modifications such as brightness, contrast and overlay graphics will also be copied. **Scanora Lite** has also feature to change language of application. To change language, select file and language and select the language from the list.